


**Now you can give your
keratoconus patients the
benefits of...**

MARCONUSTM

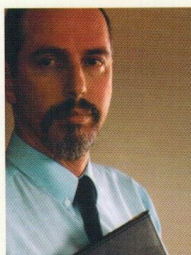

Stephen Marton has prescribed contact lenses for keratoconus for over twenty years. In conjunction with CIBA Vision, Stephen has developed MARCONUS, utilising the experience he has gained with thousands of keratoconus patients.

Apart from his practice in Sydney, he has also been on staff at Sydney Eye, Prince of Wales and Westmead hospitals, and has worked in conjunction with scores of corneal and general Ophthalmologists and Optometrists.

Marconus Features

- User friendly fitting system simplifies the keratoconus prescribing process.
- A wide range of base curves from 42.00 to 69.00 Dioptres, allowing for the prescribing of the flattest to the most severe keratoconus patients.
- A wide variety of peripheries accommodate the various keratoconus types.
- Comprehensive trial set with an extensive base curve range and two different peripheries.
- Spherical central optics which assist lens centration and visual acuity, even with varying lens positions.

Marconus Benefits


- Ease of prescribing
- Reduced chair time with a high success rate at first time fitting
- Comprehensive central and peripheral control
- Excellent centration & visual acuity
- Fast adaptation and comfort

MARCONUSTM


MARCONUS™

Examples of MARCONUS lenses and fluorescein patterns


Steepening base curve

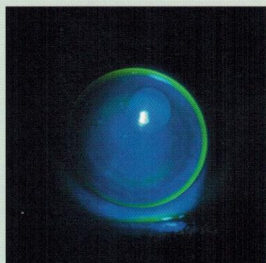

MARCONUS 2 BC 55.00D DIA 8.6mm
Heavy apical bearing

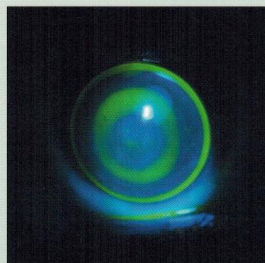


MARCONUS 2 BC 56.00D DIA 8.6mm
Apical bearing

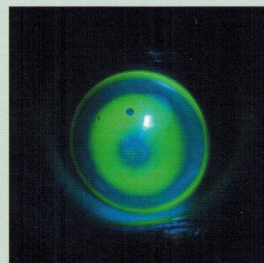


MARCONUS 2 BC 57.00D DIA 8.6mm
Moderate apical bearing


Flattening periphery

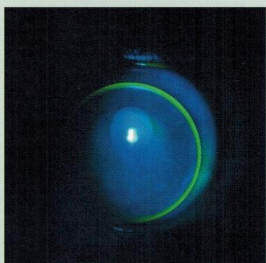

MARCONUS 2 BC 52.00D DIA 8.6mm
Narrow periphery

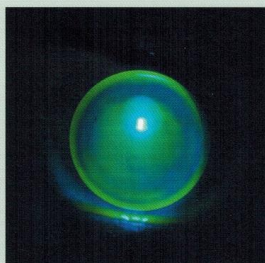


MARCONUS 3 BC 52.00D DIA 8.6mm
Ideal periphery

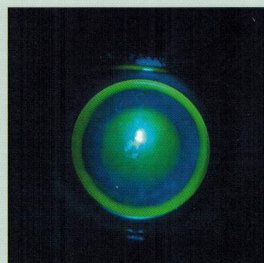


MARCONUS 4 BC 52.00D DIA 8.6mm
Generous periphery

Flattening periphery

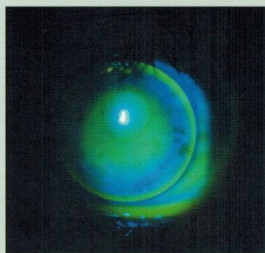


MARCONUS 3 BC 50.00D DIA 8.6mm
Ideal centre, Narrow but acceptable periphery

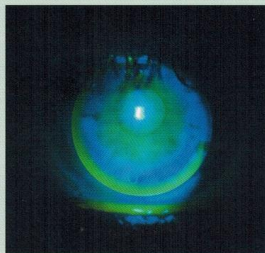


MARCONUS 4 BC 50.00D DIA 8.6mm
Ideal centre, good periphery

Example of steep fitting lens

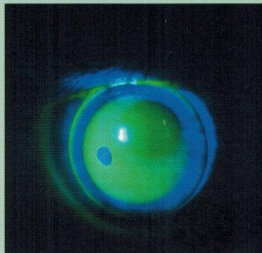

MARCONUS 2 BC 57.00D DIA 8.6mm
Steep centre, narrow periphery

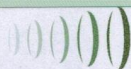

CAPRICORNIA
CONTACT LENS

TO ORDER CONTACT CUSTOMER SERVICE

TOLL FREE 1300 650 994

FAX 07 3208 8506

EMAIL info@capricorniacontactlens.com.au